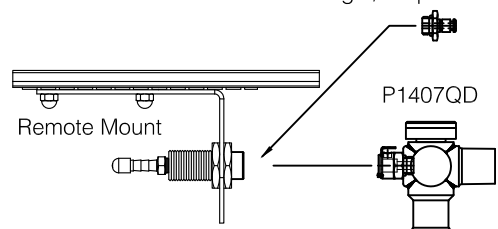
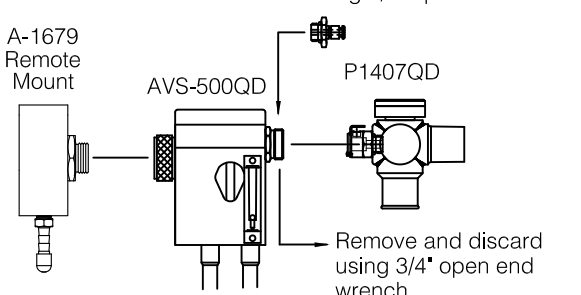
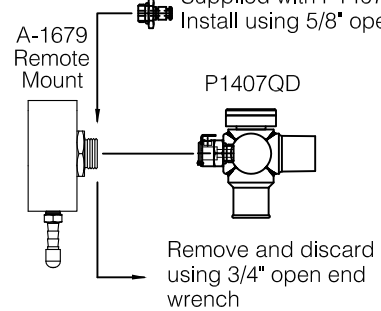
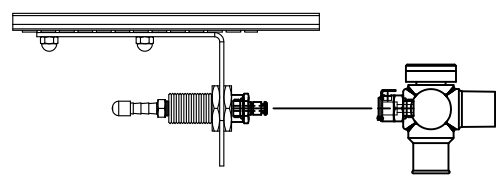
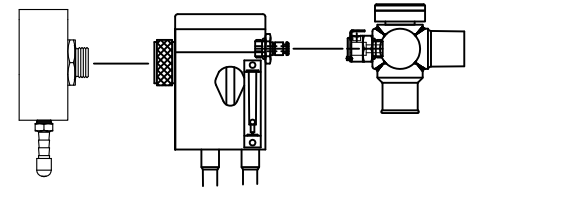
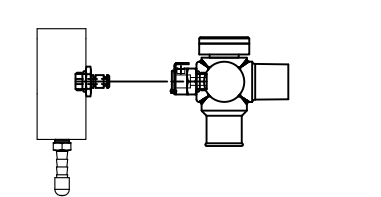
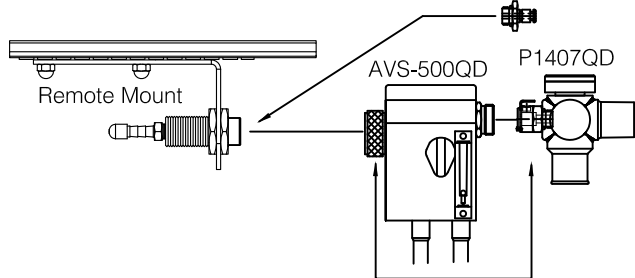
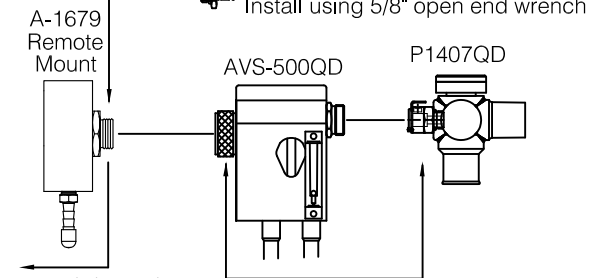
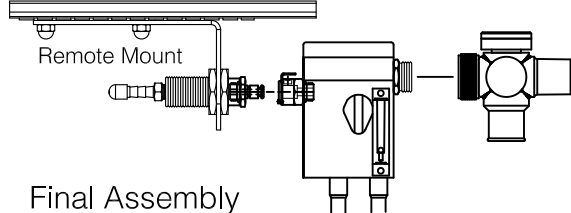
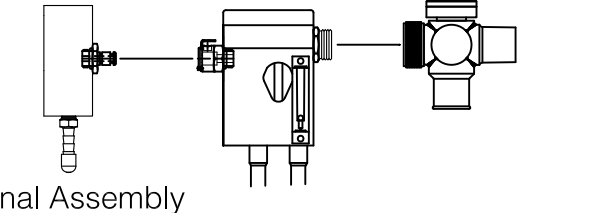


Remote Mount other than A-1679 without AVS	For A-1679 Remote Mount Only with AVS Quick-Disconnect between Bag Tee and AVS	For A-1679 Remote Mount Only without AVS
<p>Parts as shipped</p> <p>Supplied with P1407QD Install using 5/8" open end wrench</p>  <p>Remote Mount</p> <p>P1407QD</p>	<p>Parts as shipped</p> <p>Supplied with P1407QD Install using 5/8" open end wrench</p>  <p>A-1679 Remote Mount</p> <p>AVS-500QD</p> <p>P1407QD</p> <p>Remove and discard using 3/4" open end wrench</p>	<p>Parts as shipped</p> <p>Supplied with P1407QD Install using 5/8" open end wrench</p>  <p>A-1679 Remote Mount</p> <p>P1407QD</p> <p>Remove and discard using 3/4" open end wrench</p>
 <p>Final Assembly</p>	 <p>Final Assembly</p>	 <p>Final Assembly</p>
Remote Mount other than A-1679 with AVS	For A-1679 Remote Mount Only with AVS Quick-Disconnect between Remote Mount and AVS	<u>Configuring components:</u> When replacing/removing components, care should be taken to ensure that rubber seal inside of metal knob is not lost. When installing components, tighten until snug. When finished, confirm that seal inside of knob is present.
<p>Parts as shipped</p> <p>Supplied with P1407QD Install using 5/8" open end wrench</p>  <p>Remote Mount</p> <p>AVS-500QD</p> <p>P1407QD</p> <p>Swap sets of parts using 3/16: Hex Wrench &amp; 5/8" open end wrench</p>	<p>Parts as shipped</p> <p>Supplied with P1407QD Install using 5/8" open end wrench</p>  <p>A-1679 Remote Mount</p> <p>AVS-500QD</p> <p>P1407QD</p> <p>Remove and discard using 3/4" open end wrench</p> <p>Swap sets of parts using 3/16: Hex Wrench &amp; 5/8" open end wrench</p>	<p><u>Note:</u> In the quick-disconnect design, it is normal for the AVS or Bag Tee to rotate after installation is complete. Porter recommends that a standard vertical orientation be maintained during normal use (as shown on included diagrams).</p>
 <p>Final Assembly</p>	 <p>Final Assembly</p>	<p><u>How to Use Quick-Disconnect:</u> Slide AVS or Bag Tee with Quick-Disconnect female over Quick-Disconnect male until fittings make a clicking sound. To remove, depress &amp; hold metal clip on female fitting, while removing from male.</p> <p style="text-align: right;">FM-1197 Rev 0 08/11</p>