

CARPENTER'S INSTRUCTIONS


Tank Room

- 1. Must be in a separate room or cabinet.
- 2. A drywall studded room with a one hour fire rating will suffice.
- 3. Door should lock.
- 4. Room should be vented (72 square inch minimum). A vent in the door will suffice, only if no other area is available.
- 5. Attach tank restrainers with lag screws. Install 40" from the floor.
- Tank Room Instructions (LB-315) and Caution for Non-flammable Medical Gases in Use (LB-612) are posted.

Minimum size for tank room

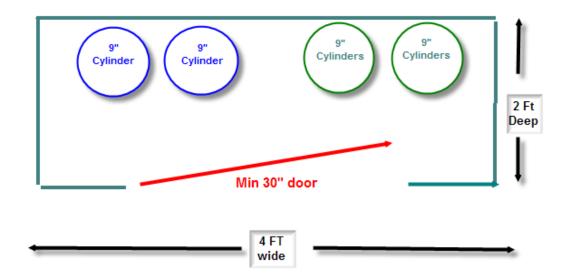

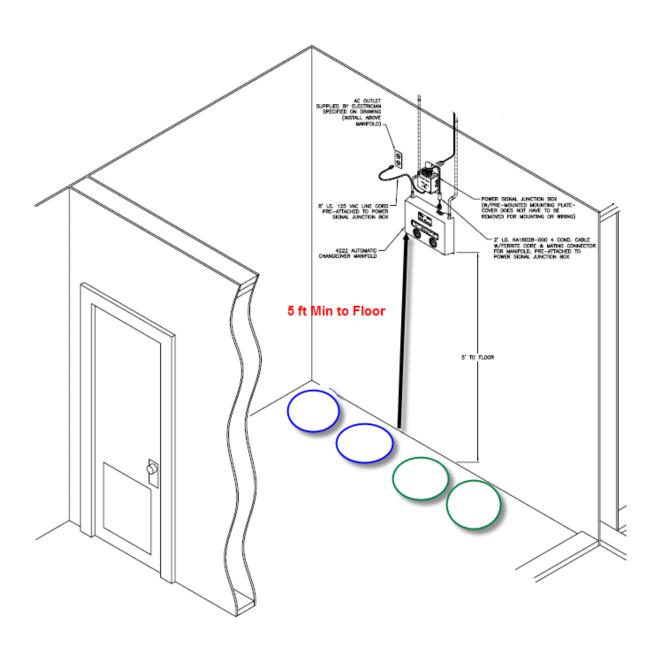

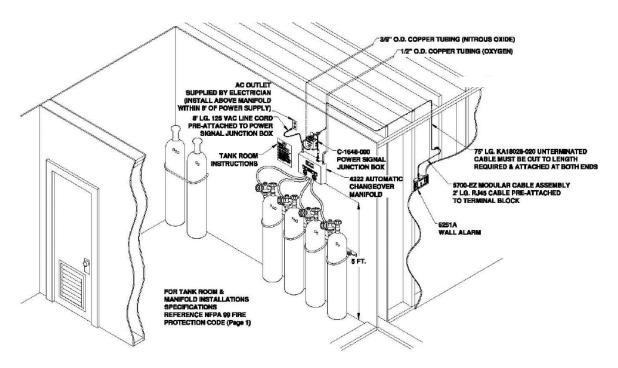



TYPICAL TANK ROOM LAYOUT