


# 9000-2 INSTALLATION

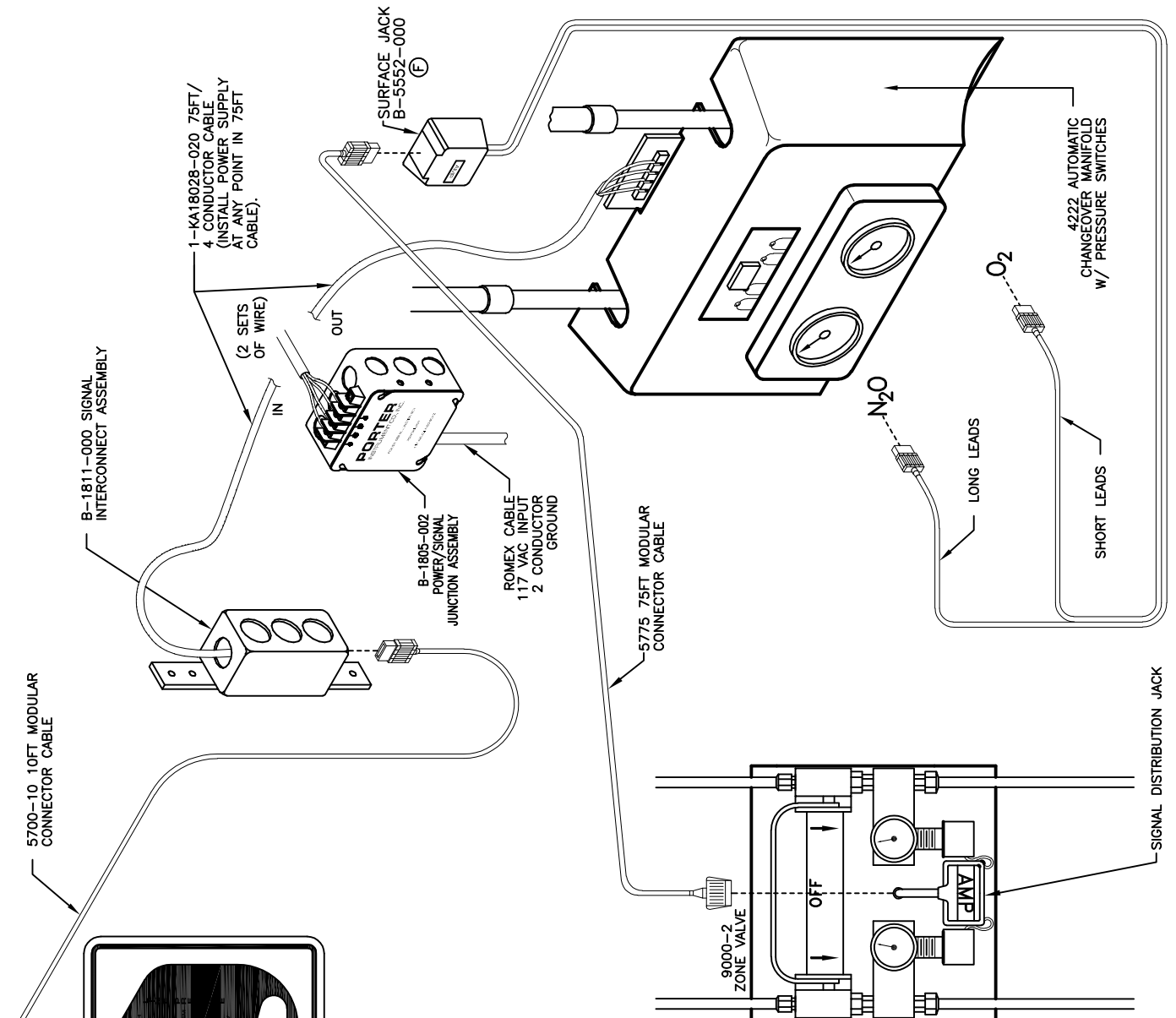

1. ZONE VALVE ASSEMBLY 9000-2 IS INSTALLED DOWN-STREAM OF MANIFOLD USING FITTINGS PROVIDED.
- ⓔ 2. MODIFY MANIFOLD (4222) TO INSTALL SURFACE JACK. Wires from HIGH/LOW PRESSURE SWITCHES AND RECONNECT LEADS FROM SURFACE JACK AS SHOWN. NOTE: ON NEWER MODELS CONNECT TO J4 ON THE O2 (RIGHT SIDE) AND TO J6 ON THE N2O (LEFT SIDE).
3. CONNECT SURFACE JACK TO WALL BY ADHESIVE STRIP ON BACK.
4. PLUG IN MODULAR CABLE #5775 TO ZONE VALVE AND SURFACE JACK.
5. FUNCTION TEST SYSTEM BY VARYING PRESSURE AND MONITORING ALARM FUNCTION FOR BOTH N2O OR O2 AT HIGH, NORMAL, AND LOW PRESSURE SIGNAL CONDITIONS.

PROPRIETARY NOTICE		REV	EC NO.	DESCRIPTION	NAME	DATE	APPVD		
THIS DOCUMENT AND THE DATA DISCLOSED HEREIN IS NOT TO BE REPRODUCED, USED OR DISCLOSED WHOLE OR IN PART TO ANYONE WITHOUT THE PERMISSION OF PORTER INSTRUMENT DIVISION, PARKER HANNIFIN CORPORATION.		REVISIONS							
		DATE JUN. 30, 1993	DR. B. DEWALD	CH.	APP. R. ROBERTSON	SCALE NONE	FILENAME: FM-622		
		Parker Hannifin Corporation Porter Instrument 245 Township Line Road Hatfield, PA 19440, USA				WIRING DETAILS FOR ZONE VALVE MODEL 9000-2		NUMBER	REV
		FM-622		SH 1 OF 1		F			